



Pump Infusion Set




~ 26 mL
~ 320 cm
(126 in)

PVC

REF 2426-0007

- Back Check Valve
- 3 SmartSite™ Y-sites

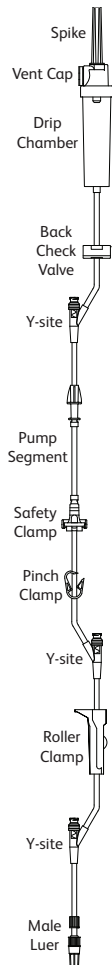

DEHP or Natural Rubber Latex are not part of the material formulation.

For more information on these symbols, please visit <http://www.bd.com/symbols-glossary>

BD Switzerland Sàrl, Route de Crassier 17, Business Park Terre-Bonne, Bâtiment A4,
1262 Eysins, Switzerland
Distributed by CareFusion San Diego, CA USA 1.800.854.7128
Made in Mexico bd.com
630-03356 Rev 00
BD, the BD Logo, Alaris and SmartSite are trademarks of Becton, Dickinson and
Company or its affiliates. © 2023 BD. All rights reserved.

Intended Use: The BD Alaris™ Pump Infusion Set is for use with the BD Alaris™ Pump Module or gravity flow and is intended for continuous or intermittent delivery of fluids and medications through clinically acceptable routes of administration.

Indication for Use: The BD Alaris™ Pump Infusion Set is indicated for use by trained healthcare professionals within healthcare facilities through intravenous, intra-arterial and subcutaneous routes for adults, pediatrics and neonates or irrigation of fluid spaces for adults.

Warnings

- To prevent free-flow, close set clamp when safety clamp on pump segment is open.
- DO NOT place set on sterile field, only fluid path is sterile.
- DO NOT use set if protective end caps are not in place.
- DO NOT use set if package is open.
- DO NOT use set if damaged or leaks.
- DO NOT use set with high pressure device (e.g. power injector). Compatible with pressure bag, up to 330 mmHg, during gravity flow use.
- Ensure fluid path is free of air before connecting to patient.
- DO NOT access y-site using a needle or blunt cannula, this will damage valve and cause leaking. Replace set immediately.
- DO NOT administer boluses < 0.2 mL via the pump; instead, deliver by IV push.
- For IV push medications, clamp tubing above y-site. DO NOT administer IV pushes through y-sites that are located above the pump when the set is loaded into the pump.

Precautions

- When gravity flow is necessary, close tubing roller clamp, remove set from pump, the safety clamp will automatically close.
- To open safety clamp, press the blue ridged release tab while sliding the white safety clamp back into the blue portion of the set.
- Verify all connections are properly secured before starting infusion.
- This set can be used for up to 96 hours and should be changed accordingly.
- DO NOT leave slip-luer tip syringes unattended when attached to y-site.
- During use of y-site, fluid may be observed between the housing and blue piston. This fluid does not enter the fluid path and requires no action.

Directions: Use Aseptic Technique

Prior to every access, swab the top of the y-site access surface for 2-5 seconds with 70% isopropyl alcohol and allow to dry.

1. Remove set from package, close roller clamp.
2. Spike fluid container and hang.
3. Squeeze and release middle of drip chamber until approximately 2/3 full.
4. Open vent cap on spike after filling drip chamber, if container requires venting.
5. Open roller clamp and adjust flow to slowly prime tubing.
6. When priming is complete close roller clamp.
7. Insert pump segment and safety clamp into pump (For proper placement information refer to BD Alaris™ Pump Module User Manual).
8. Pull back the spin collar on the male luer to properly insert the luer tip into the compatible access device and rotate luer 1/4 turn.
9. Bring the spin collar down and rotate to tighten.

Notes

- To dispose of this set adhere to local and/or other governing regulations for medical device and/or biohazardous waste disposal.

For product questions, contact your BD representative. Consult facility protocols. The CDC, Infusion Nurses Society and other organizations publish information useful in developing facility guidelines.