



Before Opening

- Assemble any supplementary components
- Wash hands
- If applicable, carefully remove old dressing and discard in appropriate receptacle.

NOTE: Components are generally arranged in order of use from top to bottom

.....

1. Open package containing sterile insertion/dressing kit and Safety Infusion Set device.
2. Remove CSR wrapped kit and place onto work surface with folds up.
3. Remove face mask from outer folds of wrap and don.
4. Remove gloves from outer folds of wrap.

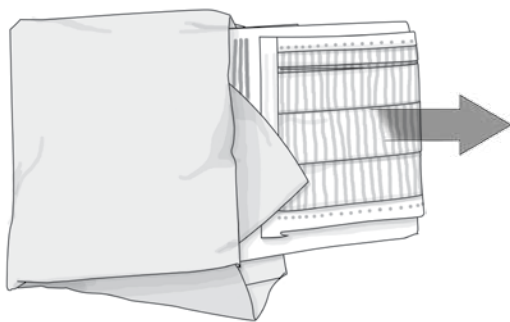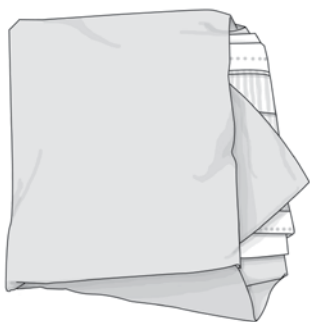

5. Aseptically unfold wrap to expose insertion dressing kit components to create sterile field.
 6. Aseptically open sterile field "SF" saline syringe and any supplementary sterile components and drop onto sterile field.
 7. Don sterile prep gloves.
 8. Aseptically drape site with sterile drape to create sterile field.
 9. Prepare insertion site using ChloroPrep^{*} Solution One-Step applicator. Consult the supplied manufacturer's instructions for details, if needed.
 10. Remove and discard gloves.
 11. Don second pair of sterile gloves.
 12. Aseptically prepare the Safety Infusion Set.
 13. Insert the needle into the port using aseptic technique. Consult the supplied Safety Infusion Set device instructions for details if needed.
 14. Secure device (use sterile gauze and adhesive strips if applicable) and apply sterile occlusive transparent dressing per institutional policy.
 15. At the end of the procedure, remove gloves and dispose of used materials per institutional protocol.
-

An issued or revision date for this guidance is included for the user's information. In the event two years have elapsed between this date and product use, the user should contact Bard Access Systems, Inc. to see if additional product information is available.

Revised date: March 2010

^{*}Bard and "Your Answer for Port Access" are trademarks and/or registered trademarks of C. R. Bard, Inc. All other trademarks are the property of their respective owners.

© 2010 C. R. Bard, Inc. All rights reserved.